

hygea® MC Card Reader - DataLogger Software

For products: hygea 1, hygea 2, hygea 2850VM & hygea 6427

1. Uninstall previous version

Uninstall the previous version of the datalogger from your PC, via control panel - add/remove programs - datalogger.

2. Download latest software version

To download the latest version of the Datalogger Software click here: <https://dl.dropboxusercontent.com/s/zknv7fmifp35u6f/C3904901-1003.zip?dl=0> or alternatively copy and paste the link into your web browser.

3. Installing the latest software

Once the file has downloaded, right click on the C3904901-1003.zip file and select 'Extract all'. When prompted, click next, and click next again, then click finish.

Double click on the setup file to run and install the new version.

4. Open the DataLogger

Double click on the UW DataLogger icon on your PC desktop and confirm the version number on the screen is 1.0.0.3. Click on 'View Cycle Results Stored on PC' to confirm the installation has been successful.

The information provided is subject to change for continual improvements